

# NIPT Sample Collection Instructions


*Please refer to Generation® NIPT Collection SOP for detailed instructions*

## Check the Form

- 1 Check that the gestational age is recorded and is more than 10 weeks at collection date
- 2 Check that the patient has read the consent information and has signed the request form
- 3 Check that the doctor has signed the request form

## Collect and Mix Sample

- 1 Draw 7-10mL of blood into the supplied **Streck tube** using the Vacutainer system  
*A minimum of 7mL MUST be collected*
- 2 *Immediately mix the tube by gently inverting 10 times*

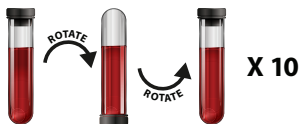

- 3 Label the tube with the patient's full name and DOB in **DD/MM/YYYY** format
- 4 Record and initial the blood draw date in the Collection Information section of the form

## Pack the Sample

- 1 Place blood tube, absorbant sheet and request form inside the supplied Biohazard bag
- 2 Place the Biohazard bag inside the Genomic Diagnostics addressed self-seal envelope
- 3 Keep at room temperature until courier collection - *Do not refrigerate or freeze*

Generation® NIPT Infoline: 1800 822 999

**Generation®**  
a new era in prenatal testing

Specialist Diagnostic Services Pty Ltd ABN 84 007 190 043 APA trading as Western Diagnostic Pathology

 **Western  
Diagnostic**  
PATHOLOGY